1 Introduction

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1 Introduction

The application allows to configure the NCBLS system by using SCS advanced configuration instead of physical configurators.

Glossary

Term	Description
NCBLS	Nurse Call Basic Level System
NCC	Nurse Call Configurator

2 Requirements

- Personal Computer with Pentium processor ≥ 2 GHz
- RAM memory: 512 MB (XP) or 1GB (Vista/7)
- SVGA graphic card with 800x600 resolution, 256 colors
- Hard disk 500 MB available
- CD-ROM reading
- Mouse
- Windows XP 32 bit Service Pack 2, Vista 32 and 64 bit, Windows 7 32 and 64 bit
- Microsoft .NET Framework 3.5


 Important note: The Microsoft.NET framework will NOT be installed automatically. It must be installed before downloading the NCC application.

3 Installation

To install this software please proceed as described:


1. Put the CD-ROM into its drive;
2. If the AutoRun is disabled, click twice on the CD-ROM icon;
3. Double click on file **NurseCallConfigurator.exe** to start the application.

The installation of the application can be performed following the setup wizard* to install the application, which must be confirmed with a single click on the button "Yes" on the Installer confirmation popup.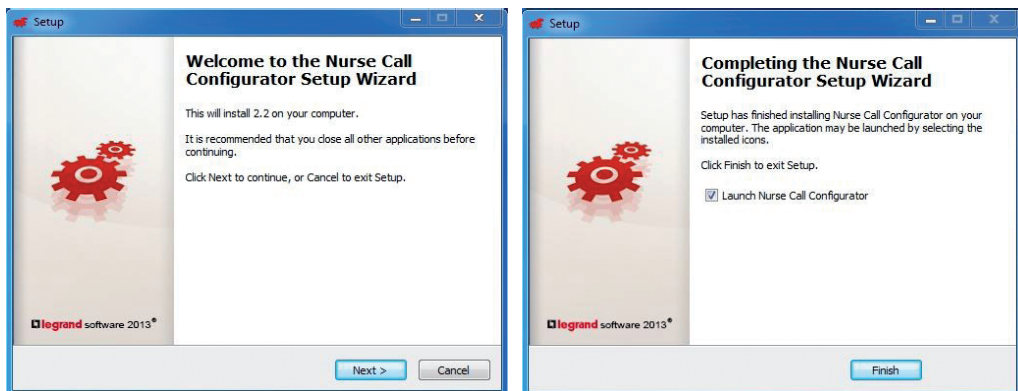

*First and last window of setup wizard

When the application starts a popup is displayed and in the main window you can see the data loading. This step needs to be completed before you can use the menu commands.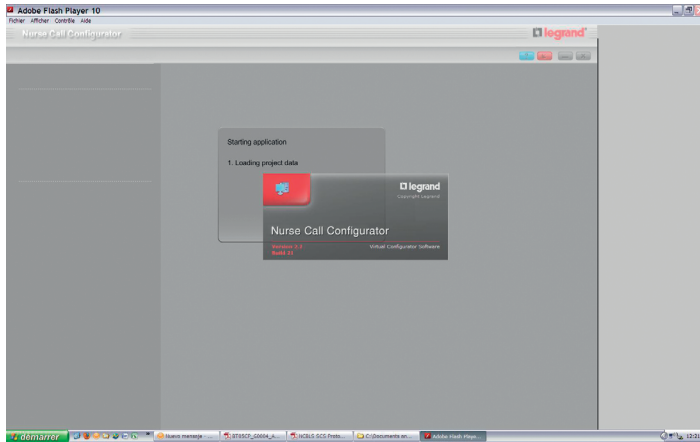

4 General Characteristics


Nurse Call Configurator allows to configure the devices of a particular system with SCS protocole (called NCBLS) by using virtual configuration instead of physical configurators. It also allows to perform system diagnostics and the testing of individual devices.

The communication between the PC and the Nurse Call system can be performed in two ways:

1. Connecting the PC to a OpenWebnetserver already present on the system;
2. Using the wired configuration kit Cat. No. 0 766 16.

NNC presents a simple user interface that is divided into three main areas:

Dedicated buttons for the general commands (e.g. "Project", "Settings", etc.), in which there is also an indicator of connection status ("Connect" or "Disconnect" button)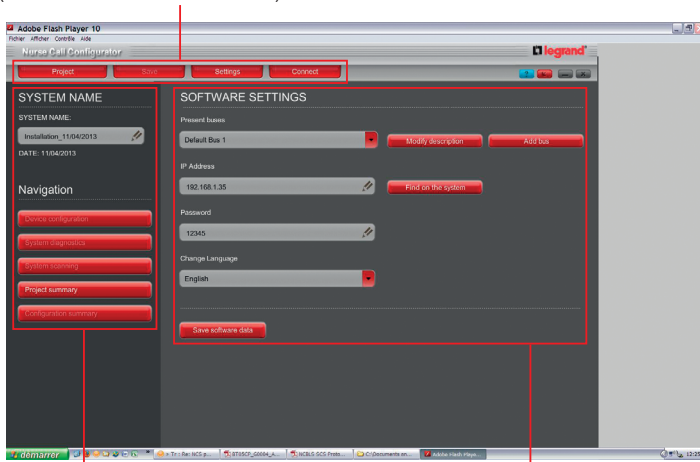

Shows the Navigation controls and the System Name

Contains central commands information in according to the selected function

As showed above, when the program is started, the screen shows the Settings menu as active. At this stage network connection has not yet been established (the connection status icon is red), and only "Project Summary" is enabled.

Press "Save software data" to save the IP address of the OpenWebnet server (by default: 192.168.1.35), password (by default: 12345) and language set.

Once a connection has been established, all functions will be enabled.